

Internal News Bulletin



INNOVATION



EXCELLENCE



RESPECT



SAFETY



TEAMWORK

Verst

LOGISTICS

Our Business Is...
An Extension of Your Business

Face Mask Removal Tips

Follow The Instructions Below

Step 1

Clean hands with soap and water or hand sanitizer before touching the mask. Avoid touching the front of the mask. The front of the mask is contaminated. Only touch the ear loops/ties/band.



Step 2a

Face Mask with Ties:

Untie the bottom bow first then untie the top bow and pull the mask away from you as the ties are loosened.

Step 2b

Face Mask with Ear Loops:

Hold both of the ear loops and gently lift and remove the mask.

Step 2c

Face Mask with Bands:

Lift the bottom strap over your head first then pull the top strap over your head.

Step 3

Throw the mask in the trash. Clean your hands with soap and water or hand sanitizer.

